# Inkubator

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09/2013

# Incubator

Dry bath incubator for BAG-BioCheck Ampoules

**Order-No.:** 7475 (adjustable to 37°C, 57°C or 60°C)

**Packaging unit:** 1

## Product features

Dry bath incubator with 14 wells for incubating BAG-BioCheck STEAM with spores from *Geobacillus stearothermophilus* at 55-60°C and BAG-BioCheck EO with spores from *Bacillus atrophaeus* at 30-35°C. Different temperatures (37°C, 57°C, 60°C) are preselected and can be changed according to the species tested.

## Instructions for Use

The incubator is designed for continuous operation and does not have a power switch. The incubator will power on when it is plugged into a power source.

1. Plug the power supply into a power source.
2. The incubator will automatically power on and perform a self-test. Upon completion, the installed firmware revision will be displayed on the LED screen for two seconds.
3. The incubator will begin its warm-up phase. During this time the configured temperature set point will fade in and out on the LED screen.
4. The incubator is ready to use when the configured set point is no longer fading in and out, and the LED screen displays the current operating temperature of the incubator

**Note:** It may take approximately 15 minutes for the incubator to warm up to operating temperature.

## Temperature Selection

1. With the incubator powered on, simultaneously press and hold the two small buttons on the rear of the incubator for ~2 seconds until the currently selected temperature set point blinks on the LED display.
2. Release the buttons, then press either button repeatedly to toggle between the available temperature set points (37°C, 57°C, or 60°C).
3. When the desired temperature set point is blinking on the display, press and hold both buttons for ~2 seconds.
4. The configured set point will fade in and out on the LED screen until the incubator has reached temperature, upon which the actual temperature of the incubator will be displayed.

## Safety precautions

- Handle with care. The surface of the incubator may become hot to the touch
- Maximum relative humidity 80% for temperatures up to 31°C decreasing linearly to 50% relative humidity at 35°C

09/2013